STERIBLEF

MOUSSE | FOAM |
SCHUIM | SCHAUM

FR Paupières et cils

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EN Eyelids and eyelashes

STERIBLEF® Foam has been specifically formulated for daily cleaning and hygiene for eyelids and eyelashes; the foam contains a special blend of essential oils from tea tree and cornflower, allantoin, provitamin B5 and vitamin E.

Properties:

STERIBLEF® Foam is a formula for topical use to effectively eliminate ocular secretions and any bacteria present on the eyelids and eyelashes.

Indication:

STERIBLEF® Foam is a medical device indicated for daily hygiene of eyelids and eyelashes, in particular in individuals suffering from **blepharitis or blepharconjunctivitis, or following eye surgery**.

Use of **STERIBLEF® Foam**, in a simple cleansing step helps to eliminate eye secretions around the eyelids and lashes.

Daily use of **STERIBLEF® Foam** helps naturally reduce risk of infection.

Ingredients:

Aqua, methylpropanediol, cocamidopropyl betaine, disodium cocoyl glutamate, panthenol, ethylhexylglycerin, glycerin, allantoin, sodium cocoyl glutamate, hexamidine diisethionate, melaleuca alternifolia leaf oil, pantolactone, centaurea cyanus flower extract, tocopherol.

Recommendations for use:

1- Apply a small amount of foam to clean fingertips or to a small compress.

2- Using circular motions, gently massage the foam over your closed eyelids.

3- Leave the foam on for 1 minute.

4- Rinse with warm water and gently pat dry.

Daily use. For more intense treatment, use twice daily for up to 7 days.

Precautions:

- For external use only. Do not apply directly into the eye.

- Wash hands before using.

- Remove contact lenses before using the foam.

- Store out of reach of children.

- Not suitable for children under the age of 36 months.

- Store in a cool, dry location below 25°C.

- Do not use if you are hypersensitive to any of the ingredients.

Adverse effects:

In the event of sensitisation or irritation, immediately stop using the product and consult a doctor or pharmacist. Any use other than that intended constitutes misuse.

Do not use if the package is damaged in such a way that the seal is not guaranteed.

Do not use after the expiry date printed on the package.

Any serious risk occurring in connection with the device must be notified the manufacturer and the competent authority of the Member State in which the user is established.

Packaging:

50 ml pump bottle to dispense foam (no gas).

Tips for daily care:

Your eyes and eyelids are very delicate and sensitive. Every day, they are subjected to a number of environmental stresses (wind, pollution, cosmetics, bacteria etc.).

These external impacts can cause inflammation of the eyes or eyelids.

Blepharitis is a chronic inflammation of the eyelids. It causes redness, discomfort, irritation and swelling. Sometimes a crust will form.

However, simple everyday care can help reduce these symptoms.

Daily hygiene:

Keeping eyelids clean is especially important for people with blepharitis or blepharconjunctivitis.

It is recommended that you use a product specifically formulated for daily hygiene of eyelids and lashes, every morning and evening before cleaning or applying treatment to your eyelids.

Good eyelid hygiene helps eliminate the crusts and external factors that can cause infections.

Massage:

Massage the eyelids using light pressure with circular motions from the inside towards the outside, across the base of the eyelashes and then from top to bottom across the upper eyelid and bottom to top across the lower lid.

The mechanical action of massage stimulates the meibomian glands and ushes away dried secretions.

Heat:

In addition to your twice-daily care and cleaning, you may wish to apply a cloth soaked in warm water to closed eyelids for 5 minutes, 1 to 2 times daily.

The heat will liquefy the hardened secretions and make it easier to remove them by massage.

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